Taiwan
Advanced
Nanotech

(Compatible with power 100V-240V)

Maelstrom 4810

User Manual

TAN Bead Nucleic Acid Extractor
(Non-sterile)

039.M1501.X61

Table of Contents

1.	Introduction	6
2.	Instrument Overview	13
3.	Installation & Get Started	16
4.	User Interface Guidelines.....	17
5.	Technical Support	38
6.	Advisory Notice	38
7.	Cleaning and Maintenance	39
8.	Disposal	39
9.	Patents	40
10.	About Manufacturer	40
11.	TANBead Instrument Warranty Policy.....	41
12.	Appendix	42

About Manual

The label on the instrument, the User Manual, and other packaging material may contain following symbols:

Catalog number

Non-Sterile

Serial number

Temp Limit
(Operating)

Manufacturer

Humidity Limit
(Operating)

The date of manufacture

Biologic Risks

Keep Away From Sunlight

Keep Dry

Consult Instructions for Use

The CE mark

In Vitro Diagnostic Medical Device

Authorized European Representative

Caution, consult accompanying documents

Hot surface, risk of burns

Watch your fingers and your hands

The symbol indicates that the product should not be discarded as unsorted waste but must sent to separate collection facilities for recovery and recycling

WARNING

- Use a power cord that meets your country's standard. In case of any questions about the power cord, contact your local distributor or reseller.
- Do not use the instrument with voltage other than the voltage specified on the device.
- Do not use the instrument with damaged power cord or a loose socket.
- When the power plug is disconnected from the AC outlet, please hold the power plug itself instead of pulling the power cord.
- Before working on maintenance, must disconnect the power plug from the outlet in advance.
- Do not pour any liquid on the instrument.
- Do not place any containers with liquid on the instrument. Doing so may cause a fire, electric shock or malfunctions of the instrument.
- Do not touch the power plug or cord if there is a chance of lightning. Failure to observe this may cause electric shocks.
- If you hear thunder or suspect approaching lightning, quickly turn off the power switch and pull the power plug from the AC outlet. Failure to observe this may cause a fire or malfunctions.

About Instrument

CAUTION

- Never attempt to remodel the instrument without the permission from the manufacturer. Doing so may lead to a fire or electric shock.
- Do not bump or knock the instrument. Doing so may cause malfunctions.
- Repairs to the instrument must be performed by agencies authorized by Taiwan Advanced Nanotech Inc.
- Only original spare parts from Taiwan Advanced Nanotech Inc can be used on the instrument.
- If the equipment is used in a manner not specified by the manufacturer, the protection given by the instrument may be impaired or invalid.


1. Introduction

Maelstrom 4810 is an automated nucleic acid platform designed for mid-to-high throughput applications. Specialized spin tips enable super mixing efficiency of magnetic beads and larger processing volume. With intuitive interface and flexible program, Maelstrom 4810 can boost molecular diagnostic laboratory productivity by transforming routine operations into a walk-away solution.

Operation Principle

Maelstrom 4810 uses magnetic rods to collect and transfer magnetic beads and spin tips to mix suspension. Rotating the spin tips will mix suspension very well. Extended magnetic rods can collect magnetic beads efficiently.

Purified nucleic acids can be obtained after cell lysis, nucleic acids absorption, wash and elution.

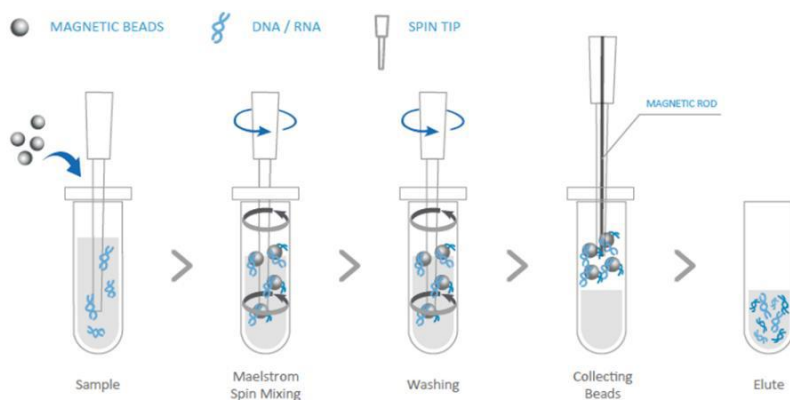1. Introduction

- The gear used in the TANBead Nucleic Acid Extractor is a hollow gear, which allows the magnetic rod to pass through the connecting portion to insert into the spin tips to perform the magnetic effect. In addition, the gear comprises an extended hollow bearing, such that the collimation of the stirring device is increased.
- The stirring device performs mixing by using the characteristics of rotation, biggest flux per unit area can be achieved compared to up and down
- The rotation stirring design can decrease the air mixed into the sample and avoid problem of cross contamination in experimental process.

Intended Purpose

- Maelstrom 4810 automated nucleic acid extractor is an *in vitro* diagnostic device, which is used to extract and purified nucleic acids from human and biological samples. The purified nucleic acid can be used with any downstream application employing PCR-based qualitative, semi-quantitative and quantitative assays. This product should be operated by technicians, physicians, and biologists with well-trained in molecular biological techniques. It is suitable for all population in human.
- Maelstrom 4810 automated nucleic acid extractor is equipped with different types of TANBead Nucleic Acid Extraction kits, which can extract and purify nucleic acids from biological samples such as blood, cfDNA, FFPE, virus, tissues, fungi, forensic, bacteria, stool and pathogen. Please refer to the following for types of specimens required.

1. Introduction

Sample Types

- Maelstrom 4810 Nucleic Acid Extractor is used with TANBead nucleic acid extraction reagent kit. Please refer to the following for sample type and compatible reagent kits for reference. Please refer to the instruction of TANBead Nucleic Acid Extraction kit

Sample	Reagent kits		
Blood	61E series	611 series	621 series
cfDNA	622 series		
	61C series		
Virus	615 series	635 series	665 series
	685 series	61H series	
Bactria	61G series	61G-SE series	6MB series
	6BG series		
Tissue	6T2 series	6K2 series	
FFPE	61P series	62P series	
Stool	6SC series		
Plant	613 series	613 series	6K3 series
Fungi/ yeast	61F series		
Environment	6EM series		
sample	6GM series		
Food and feed	6TF series		
Forensic			
Plasmid	6PE series		

1. Introduction

Environmental Requirements

To avoid shortening of the lifespan of the instrument, use Maelstrom 4810 in a location that meets the following criteria:

- Choose a location that has good air circulation.
- Place Maelstrom 4810 on the table capable of bearing at least the weight of Maelstrom 4810 ($\approx$ 45 kg).
- Do not use Maelstrom 4810 in a location where there is large temperature variability or high humidity.
- Suggested operation condition:
Temperature: 10-35°C
Relative humidity: 40-80%
- Storage and transportation condition:
Temperature: 5-60°C
Relative humidity: 20-90%
- Maximum operating altitude: 2000 m

1. Introduction

Safety Instructions and Guidelines

- This device can be used with potentially biohazardous materials. Use appropriate personal protective equipment (gloves, safety goggles, lab coat, etc.) for handling and disposing of biohazardous materials.
- Under normal condition, sound pressure level from Maelstrom 4810 does not exceed 80 dB and cause a hazard. Please contact technical support in case of higher sound pressure level.
- This device can be hazardous due to the use of chemical and biohazardous substances.
- Users should adhere to their institutional guidelines for the handling and disposal of all infectious substances used with this device.
- It is important to clean the device after every use. If samples or reagents have been spilled, it is important to clean the instrument immediately to avoid damage or contamination of samples.
- This device is to use with the compatible spin tips. Using incompatible spin tips may cause poor extraction performance.
- Read this user manual in its entirety prior to operating the device. Failure to read, understand, and follow the instructions in the manual may result in damage to the device, injury to laboratory and operating personnel or poor performance.

1. Introduction

- This instrument contains batteries compliant with the European Union Battery Regulation (EU) 2023/1542. The battery meets chemical restriction limits, is CE-marked in accordance with the Regulation, and is subject to the EU take-back and recycling obligations.

Safety Requirements

This medical device has passed the tests and conformed to the following standard:

- The device has passed the tests and conformed to the standards of IEC 61010-1:2010+A1:2016 (Edition 3.1) and EN 6010-1:2010+A1:2019, "Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 1: General requirements".
- IEC 61010-2-101:2018 with IEC 61010-1:2010 + A1:2016 and EN 61010-2-101:2017 with EN 61010-1:2010 + A1:2019, "Safety requirements for electrical equipment for measurement, control and laboratory use - Part 2-101: Particular requirements for in vitro diagnostic (IVD) medical equipment".

EMC Requirements

The medical device has passed the tests and conformed to the following standards:

- IEC 61326-1:2020/ EN IEC 61326-1:2021 Electrical equipment for measurement, control and laboratory use - EMC requirements - Part 1: General requirements
- IEC 61326-2-6:2020/ EN IEC 61326-2-6:2021 "Electrical equipment for measurement, control and laboratory use - EMC requirements - Part 2-6: Particular requirements - In vitro diagnostic (IVD) medical equipment".

1. Introduction

Equipment list

Please note that following accessories accompanying with instrument may vary from region to region.

Power cord x1

Spin Tips Assembled Box

Magnetic separator x1

=Remark=

Power cord varies from region to region.




2. Instrument Overview

ITEM	SPECIFICATION
Model	Maelstrom 4810
Weight	45 kg
Dimensions	580(W) x 430(L) x 470(H) mm
Power rating	AC100-AC240V, 50/60Hz, 2.5-5A
Fuse	250V, 5A
Max. Throughput	48 samples per run
Process. volume	50µl -1,600µl
Spin speed	up to 3000 rpm
Heater	12 independent heating blocks
Magnetic rod	>3900 gauss
Display	7-inch touchscreen
UV	UV-C type, 8W
HEPA	E 10 class
Program Name	Firmware: C51; Adjustment Program: C++; APP: C#
UDI-DI	04719880129195(E.U.); 04719880129171(U.S.A) 04719880128969(TW)

2. Instrument Overview

Front and side view

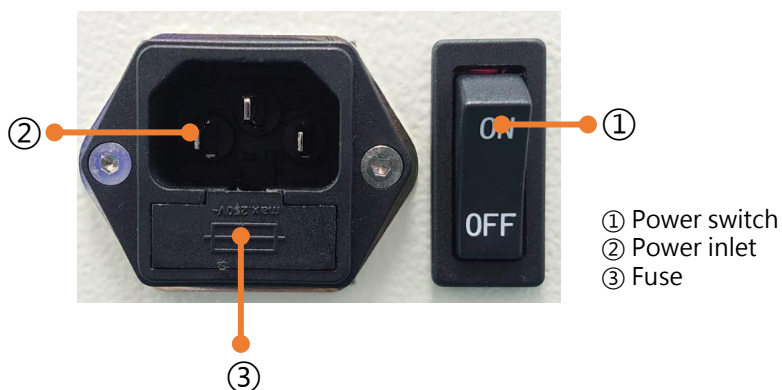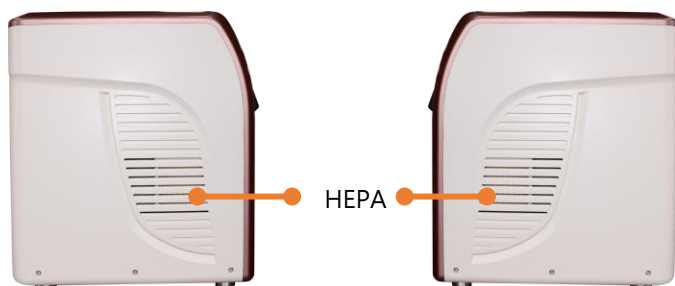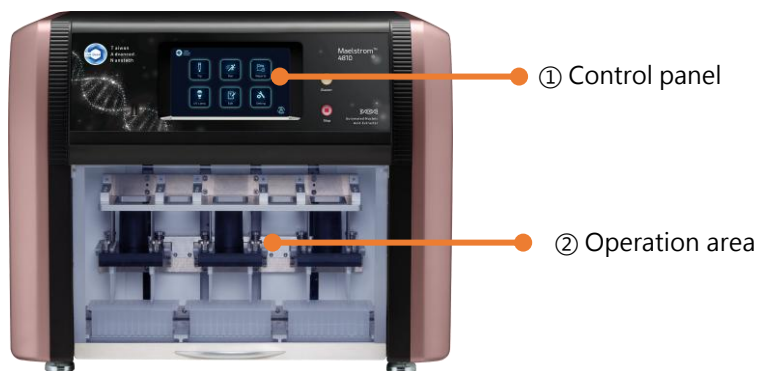2. Instrument Overview

Control Panel

To turn off the alarm

Emergency to stop machine

Please note that only TANBead technicians and authorized technicians or service engineers shall engage in program upgrade.

3. Installation & Get Started

Installing Maelstrom 4810 is easy. Please follow the guide below to correctly set up the instrument.

Please note that the weight of instrument is around 45 kg, it is highly recommended 2 or more people to handle it when you take it out from the box.

Step 1:

Take the instrument out from the box. (2 or more people is highly recommended)

Step 2:

Remove the EPE around the instrument.

Step 3:

Place the instrument to a flat-table that can bear at least 70 kg.

Step 4:

Connect the power supply cable to the power inlet. (The instrument operates at voltages of 100-240 VAC and the frequency range of 50/60 Hz)


Step 5:

Remove the EPE inside the operation area before turning on the power switch.


4. User Interface Guidelines

Enter the user code

Enter the user code to access the instrument. The default code is 333. Please keep 333 just in case you forget it. You can create a new user code on the setting page.

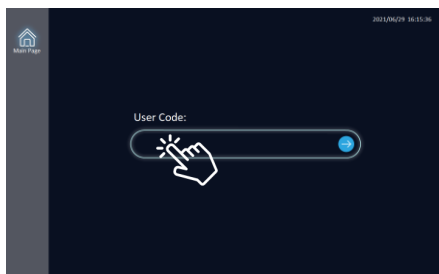
Press the blank space to enter the user code.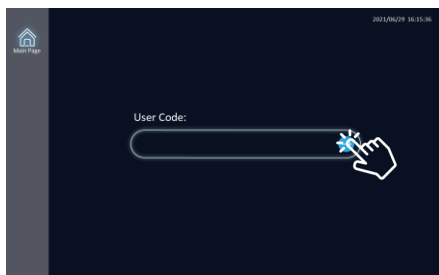

Press the blue arrow icon to start the procedure.

* If user code is not existed, "User code invalid" will show upon the column with red color.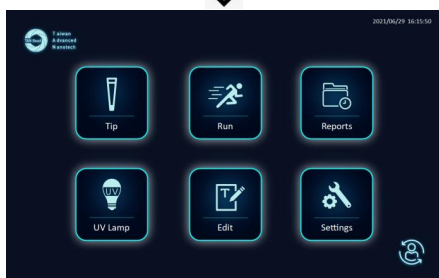


You will be leaded to the main page after the successful registration.

4. User Interface Guidelines

Main Page

There are six functions that can be used on main page.

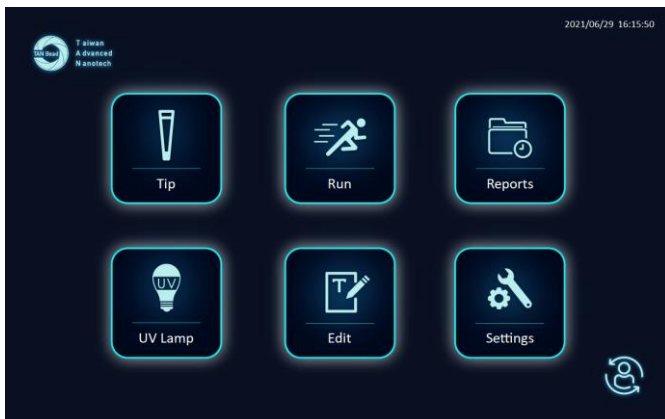To mount and eject spin tips

To execute the extraction program

To export extraction records

To use UV light to clean the instrument

To add/delete/edit/import/export extraction program


To set basic parameter of instrument

4. User Interface Guidelines

Tip (Mount and Eject Spin Tips)


Please complete the tip mounting before you start the extraction protocol.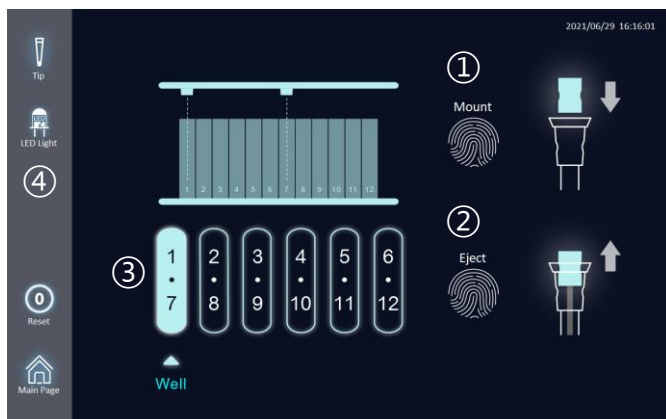


- ① **Mounting tips.** press the fingerprint icon, the tips will be loaded on the assigned lane which show on the ③ .
- ② **Ejecting tips.** press the fingerprint icon, the tips will be ejected on the assigned lane which show on the ③ . Please be noticed that the lane you choose to eject the tips should be EMPTY.
- ③ **Mount / Eject assigned lane.** The tips in these lanes will be mounted / ejected after we act.
- ④ **LED light.** press this icon will light up the operation area.

*Please mount tips separately in each plate site. You will need to operate tip-mounting function 3 times to complete all 48 tips mounting.

4. User Interface Guidelines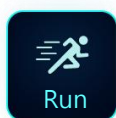

Run function is designed to execute nucleic acid extraction program stored in the instrument.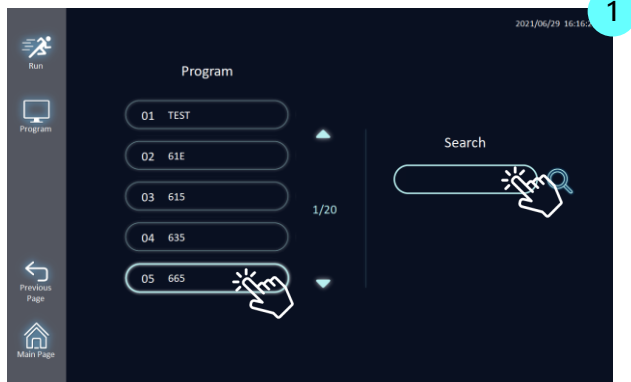

- ▲ Maelstrom 4810 is capable of storing up to 100 programs. Please follow the guideline of reagent kits and correctly select the program. Or you could type the key words in the search slot to save the searching time.


- ▲ Kit and Tip lot numbers are not a must for executing the program, but we highly suggest you input them for further tracking. Press the Run icon twice to proceed the program.

4. User Interface Guidelines


Run (Execution of Extraction Program)

Step 3: Input sample ID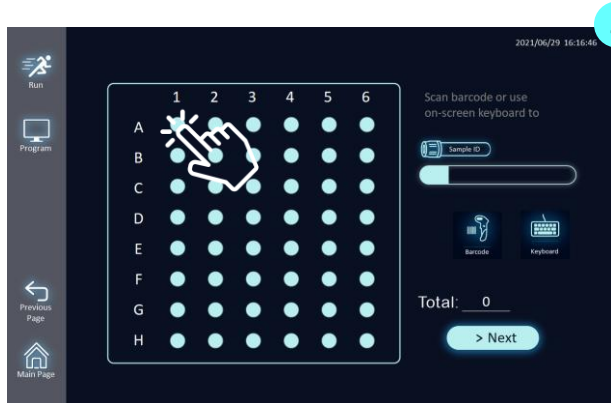


- ▲ press the circle first and then use on-screen keyboard or USB barcode scanner to scan in if 1-D barcode is available. Sample ID can bear 18 characters input. Once completed, press [Next] to continue.

4. User Interface Guidelines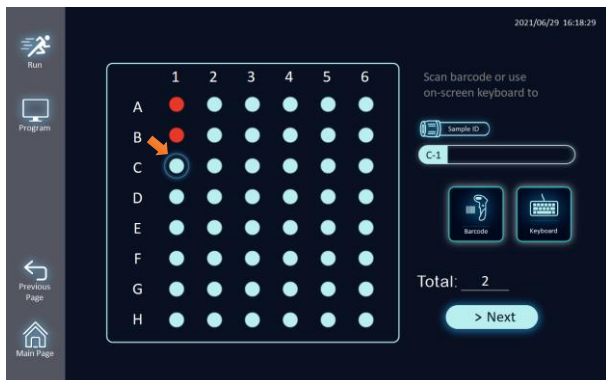

- ▲ The circle will turn red after the sample ID input successfully. The input order will automatically turn to next number, e.g. A1 will turn to A2.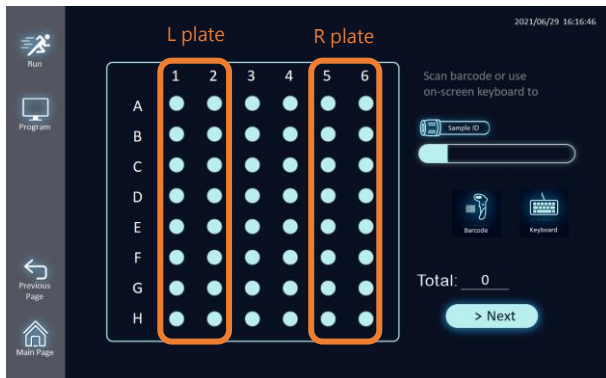


- ▲ Row 1 and 2 stand for the Well 1 and 7 accordingly of left plate. Row 3 and 4 stand for the Well 1 and 7 accordingly of middle plate. Row 5 and 6 stand for the Well 1 and 7 accordingly of right plate.

4. User Interface Guidelines

Run (Execution of Extraction Program)

Step 4: pre-heating and plate type selection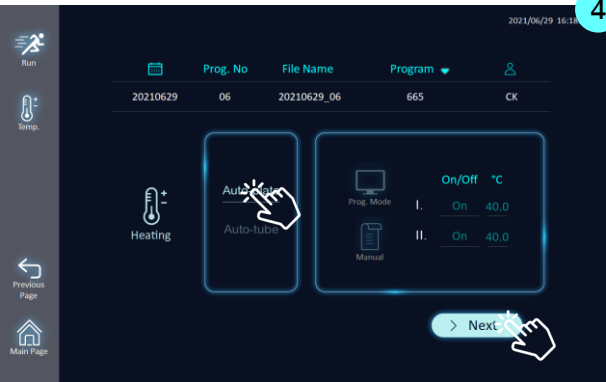

▲ The plate type should be chosen in this page. The type you chose will affect the auto-eject tips or not.

The pre-heating temp. The status is fixed (on, 40°C). If you need to change the default status or value, please contact your supplier.

When the setup is ready, press the [Next] to continue.

Step 5: Ready to run the extraction program



▲ Click [Yes] to begin the operation. If you choose [No], the process will go back to previous page.


4. User Interface Guidelines

During the Run

Understanding every symbol in the operation process is important. It helps you clearly know the status of nucleic acid extraction.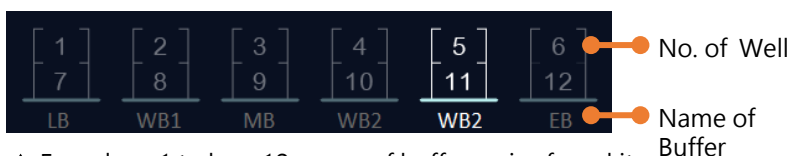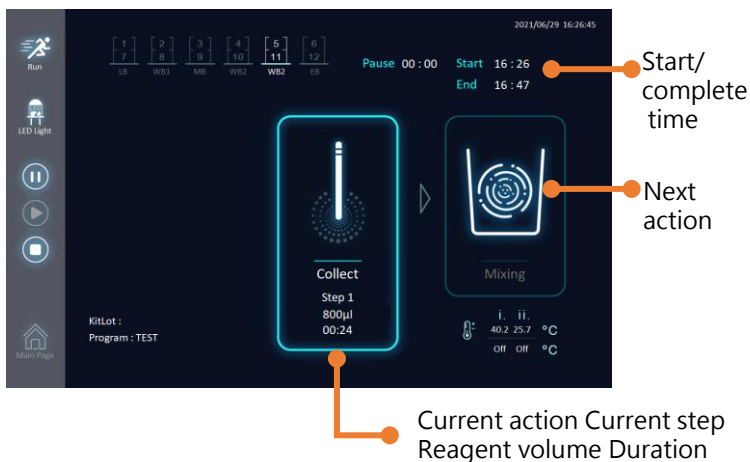

▲ From lane 1 to lane 12, name of buffer varies from kits to kits

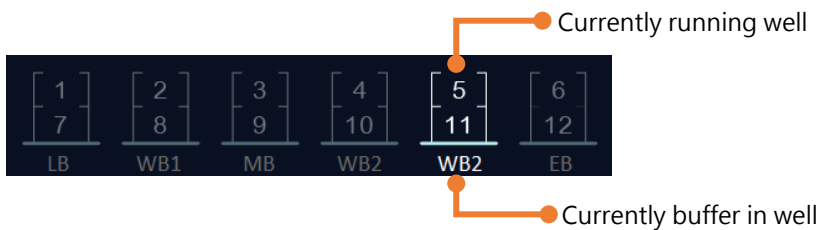4. User Interface Guidelines

During the Run

Understanding every symbol in the operation process is important. It helps you clearly know the status of nucleic acid extraction. Once the run complete, HEPA will activate one minute automatically. If you need to adjust the time, please contact your supplier.

Mixing

Beads collection

During beads collection, whether you press Pause or Stop button, the instrument will not act immediately until the collection process comes to the end.

Vaporizing EtOH

Before entering Elute process, evaporating EtOH is a MUST. Poor evaporation on EtOH will influence the nucleic acid product quality significantly.

Program completed

When a run comes to the end, the Buzzer will ring up few seconds.

Stop, Suspend the extraction program forcibly

Pause, Cease an extraction program temporarily


Resume, if you click Pause to temporarily stop the program, you can click Resume to continue the operation.

4. User Interface Guidelines

Report (Export extraction records)

Report function enables you to export/delete experiment records and the identity that performed the experiment.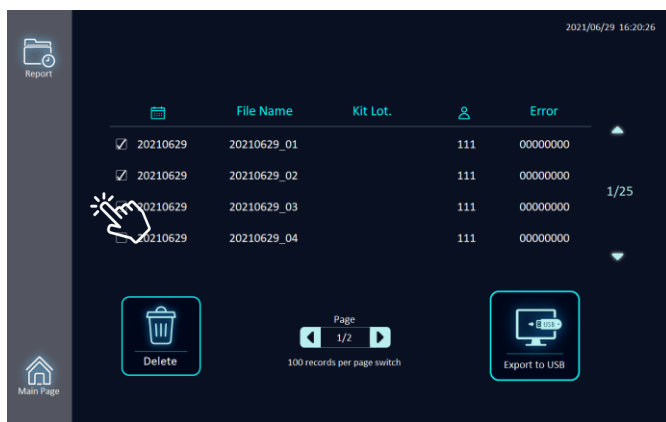

- ▲ The record could be chosen by pressing the square in front of date. The chosen record can be deleted or exported to USB.

4. User Interface Guidelines

Instrument cleaning

It is highly advised to use UV to clean the instrument after each experiment.

▲ Set UV lighting time first, and then press bulb icon to start

- ❑ Step 1: Set lighting time by clicking UV duration field
- ❑ Step 2: Click bulb icon to begin UV lighting


⚠ Before turn on UV, please must close the door of operation area. The recommended lighting time is at least 30min. The more, the better.

4. User Interface Guidelines

Edit (Edit / Delete / Copy / Import / Export Program)

Edit function enables you to customize the extraction protocol.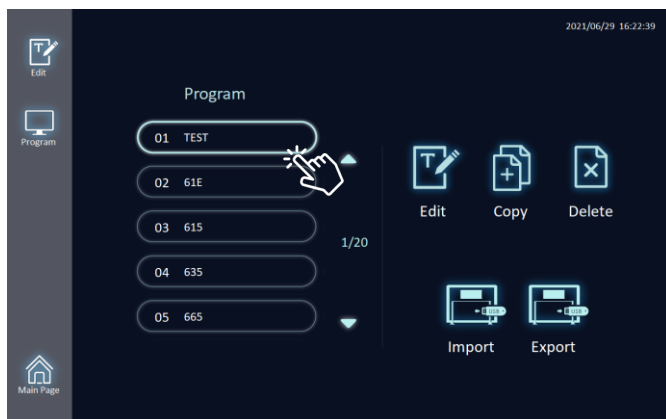

- ▲ To create or modify a protocol, it is necessary to add steps and define the parameters related to the step.

	To adjust the setting of existing protocols, which contains well volume, preheat temp, spin direction, extraction steps, time of mixing and collection.
	To add a new program, you need to use Copy to duplicate a program first, and then use Edit to write the protocol.
	Delete a program no longer used to save the memory space. Maelstrom 4810 can store up to 100 programs and each program can support totally 70 steps.
	User could import the program into the instrument by USB.
	Export function allows user to output the program by USB.

4. User Interface Guidelines

Edit a Program

▲ Click above interface to exchange the setting.

Program	Name of program, up to 10 characters
Vol. (μl)	Total processing volume, from 50-1600 μl
Heating(°C)	Heating the lane to a particular temp before mixing step Preheat temp range from 40°C to 130°C
Action	Way of binding during the mixing process For.: Forward spin (clockwise) Rev.: Reverse spin (spin direction switch regularly) For. U/D: Forward spin with up-and-down move Rev. U/D: Reverse spin with up-and-down move
Name	Name of buffer in each well LB: Lysis buffer WB1: Wash buffer 1 WB2: Wash buffer 2 MB: Magnetic beads WB3 : Wash buffer 3 EB: Elute buffer

4. User Interface Guidelines

Edit a Program


▲ Touch the screen to edit the protocol.

Step	Extraction step Maelstrom 4810 supports up to 70 steps in a program
Well	Number of well
Temp	Please adjust to appropriate protocol temperature before running the test which is suitable for different reagents. Protocol temp and Preheat temp are different. When executing a program, preheat temp will be fulfilled in advance, and protocol temp will be acted as the target plate is about to be used.
Time	Time of mixing, range from 0.0 to 60.0 mins
Speed	Speed of mixing, range from 500 to 3500 rpm
Collect	Time of collecting beads, range from 0.0 to 60.0 mins
Vapor	Time of evaporate EtOH, range from 0.0 to 60.0 mins
Pause	Will automatically pause after executing the current step. If additional reagents need to be added during the extraction process, this step can be taken.

4. User Interface Guidelines

Add a new program

1. Touch one of standard programs button.
2. Press Copy button and fill in blank space.
3. Choose new program and press Edit button to set the relevant parameters.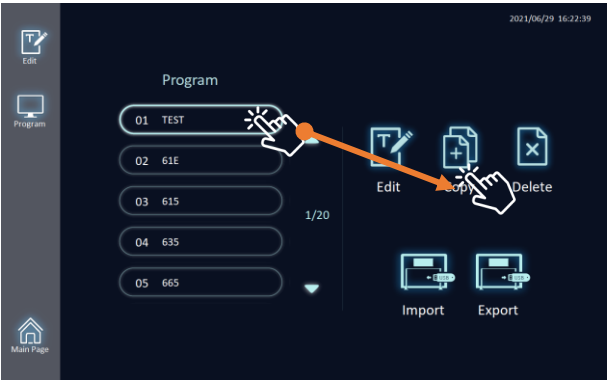

- ▲ Touch one of standard programs button and press Copy button.


- ▲ Confirm the number to add the program. The keyboard will appear after press on destination number. Once complete, click Yes to add the program.


4. User Interface Guidelines

Delete a Program

To delete a program, press the program no longer need and then click Delete icon to execute.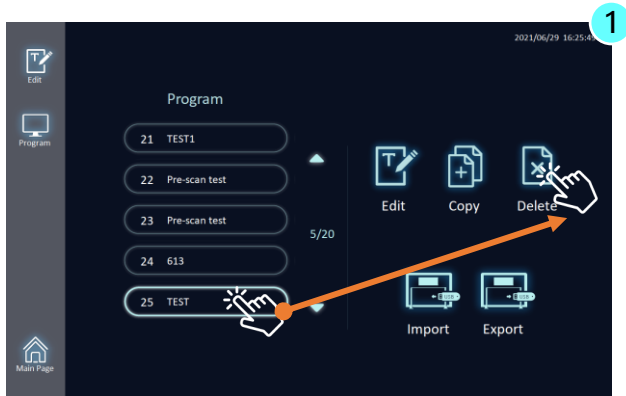

▲ Tab the unwanted program, and tab the Delete icon

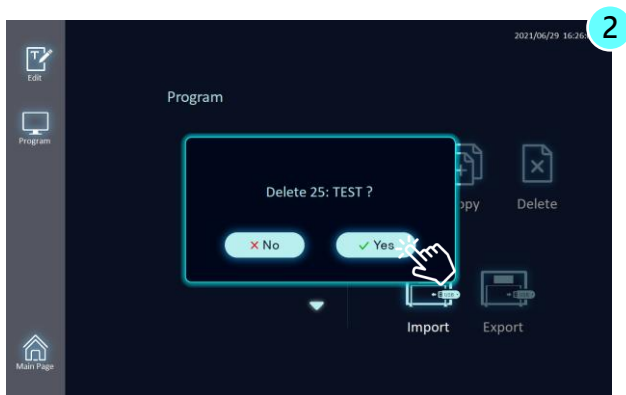
▲ Click Yes to confirm the deletion

4. User Interface Guidelines

Setting

Setting is designed for user to change basic system parameter setting, such as display language, time info and barcode scanner management.

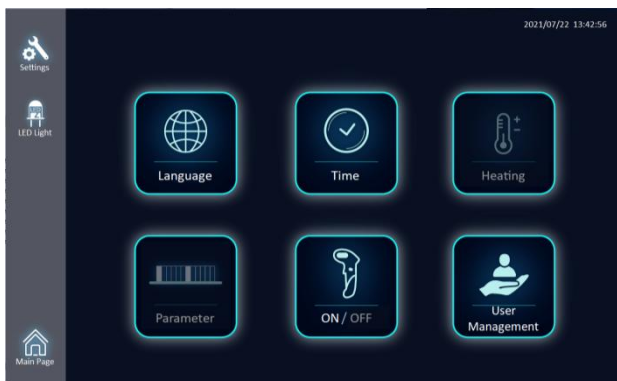- ▲ Admin user is allowed to assess Language, Time, user and Barcode functions.

Language

Maelstrom 4810 supports multi-language display, including English, Traditional Chinese, Simplified Chinese, Korean and Spanish.

Time


When a new instrument is delivered, please adjust to the right time zone.

Barcode

Scan code to input related information. e.g. lot number, sample ID.

User management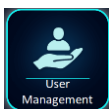


The identity of the user. This function allows users to create their own code to secure the experiment data.


4. User Interface Guidelines

Setting (User management)

Log in by default code 333 or create new user code in this function. The new user code has the same permissions as default code and each user could only view their code and code 333.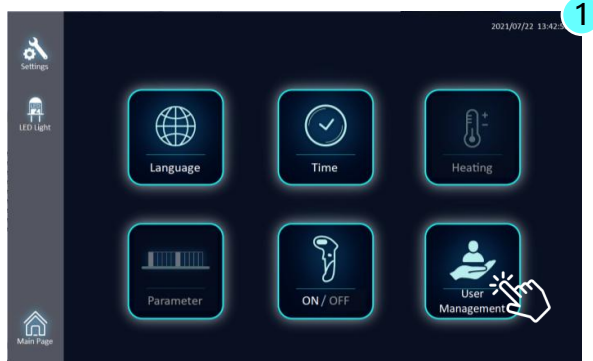

▲ Press User icon to access.

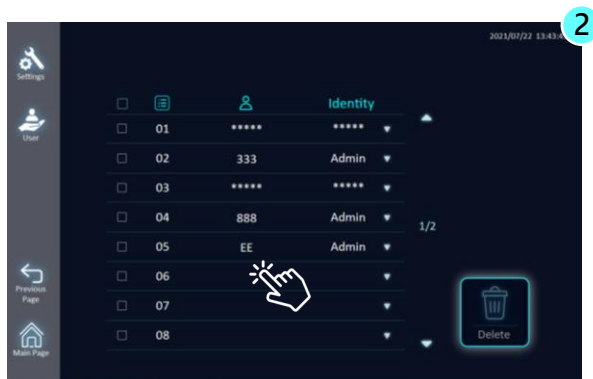
▲ Press the blank space to create a new user code. The new user code has the same permissions as default code.

4. User Interface Guidelines

Setting (Language)

Maelstrom 4810 supports multi-language operating interface including English, Traditional Chinese, Simplified Chinese, Korean and Español.

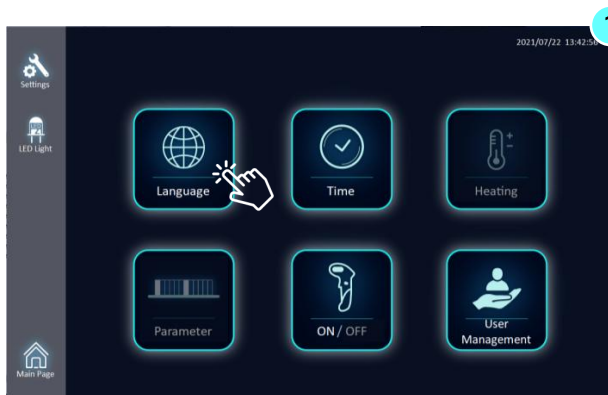▲ Press Language icon to access

▲ Choose the different languages

4. User Interface Guidelines

Setting (Time)

Set up the correct time zone.


▲ Click icon to set Date and Time for the system

4. User Interface Guidelines

Setting (Barcode)

Maelstrom 4810 can be compatible with USB-type infrared barcode scanner. When the barcode scanner is correctly connected to the machine, the code scanning function will be automatically turned on. This machine can use barcode to read and write data after clicking the corresponding well plate on the operation page.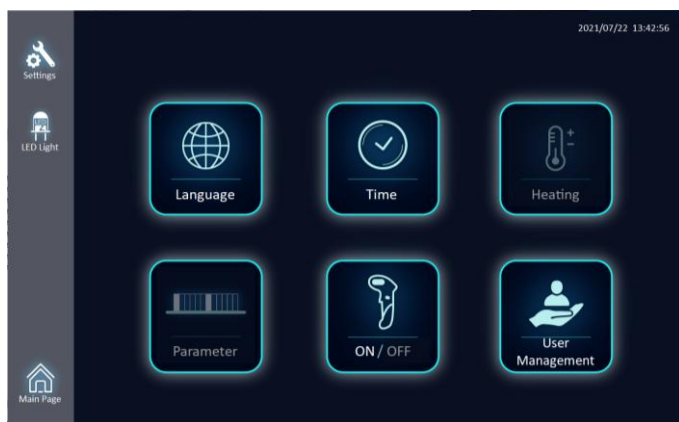

▲ Barcode scanner function

- ⚠ When the user scans the wrong barcode or needs to change the barcode information, click the corresponding well plate on the screen and enter the keyboard mode to complete the manual deletion, and then execute the barcode scan again.

5. Technical Support

Taiwan Advance Nanotech Inc. provide post-sales service and technical support. In case of any question, please try to contact our authorized distributor nearest to you or our company for more help.

If there is a need to contact us, you can call our rep number at +886-3-3167568 or email us at service@tanbead.com for further assistance.

To efficiently solve the problems and answer your questions, please provide us your instrument serial number when you talk to our technician personnel. Thanks for your cooperation.

6. Advisory Notices

The Company may issue advisory notices from time to time (including safety alerts, reinforced instructions for use, or corrective recommendations) to ensure continued safe and effective use of the product.

- For the latest advisory notices, please contact Customer Service service@tanbead.com
- The Company will retain records of distribution and acknowledgment of issued advisory notices and will notify distributors / end users as appropriate.

7. Cleaning and Maintenance

- It is important to clean the device after every use. If samples or reagents have been spilled, it is important to clean the device immediately to avoid damage or contamination of samples.
- Wear gloves and appropriate personal protective equipment. If the device is used with biohazardous materials, dispose of any cleaning materials used in accordance with your institutional guidelines.
- The device may go through a run with the magnetic rods unprotected. If this happens, the magnetic rod needs to be cleaned immediately.
- To clean the magnetic rods, wipe with a soft cloth dampened with pure water. Do not use alcohol solvent.
- If the magnetic rods cannot be cleaned, please contact TANBead (Tel: +886-3-3167568) for technical assistance.

8. Disposal

In case the product is to be disposed of, the relevant legal regulations are to be observed. As disposal regulations may differ from country to country, please contact your supplier if necessary. Please follow your institutional and country-specific requirements to handle the disposal of accessories. Device must be decontaminated prior to disposal.

9. Patent


Patent List	
USA	US09616398B2
EU	EP2937136
Canada	CA2862946
Japan	JP6151735B2
Korea	KR101696517B1
China	CN104971638B
Taiwan	TWI526245B
WIPO	WO2016127292

10. About Manufacturer

Here comes the detail of instrument manufacturer and its EC Rep.

- **Manufacturer** : Taiwan Advanced Nanotech Inc.
- **Head Quarter Address**: 6F, No. 188, Wenhe Rd., Guishan Dist., Taoyuan City 333, Taiwan
- **Manufacturer Address** : 4F, No. 188, Wenhe Rd., Guishan Dist., Taoyuan City 333, Taiwan
- **Manufacturer Tel** : +886-3-3167568

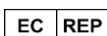- **Contact details** : mdi Europa GmbH
- **Address** : Langenhagener Street 71, D-30855 Langenhagen, Germany
- **Telephone** : +49-511-3908-9530

11. TANBead Instrument Warranty Policy

This warranty sheet covers the Nucleic Acid Extractors manufactured by Taiwan Advanced Nanotech Inc. (Hereinafter referred to as TANBead).

TANBead warrants that under normal use conditions, this product will not have any material or manufacturing defects for one year from the date of purchase. If any defect is found within the warranty period, TANBead will repair or replace the product free of charge.

This warranty policy does not apply to the following situations:

- The product is damaged due to improper use, operation, storage, maintenance, abuse, or transportation of the products.
- The product is damaged due to accident, disaster, natural phenomenon, or other force majeure factors.
- The product is modified, disassembled, reassembled, or repaired by unauthorized personnel.
- The product exceeds its expiration date or is not used according to the instructions.
- Damage or loss caused by factors beyond TANBead's control, such as sample quality, experimental conditions, or user error.

To claim the warranty, please contact our customer service department with the following information:

- Proof of purchase (invoice, receipt, etc.)
- Product name (model) and serial number
- Description of the problem and evidence of defect or malfunction (photos, test results, etc.)

We will provide you with instructions on how to return the products. Please do not return the products without our authorization. TANBead will bear the shipping cost of returning the product, but not any other expenses or losses.

This warranty policy is your only remedy and replaces any other express or implied warranties or conditions. In no event shall TANBead be liable for any indirect, special, incidental, or consequential damages, including but not limited to loss of profits, business interruption, data loss or other commercial losses.

We appreciate your business and hope that you are satisfied with our products. If you have any questions or concerns, please feel free to contact us at any time. Thank you for choosing TANBead! If you have any questions about our warranty services, please email to service@tanbead.com

To register your TANBead instrument for more technical support and services, please sign up an official membership of TANBead.

12. Appendix

Fuse Replacement on Maelstrom 4810

Fuse Specification:

- UL recognized component
- Rated 5A, 250VAC x2pcs
- UFE, Size 5.2Φ x 20mm

Safety precautions of replacing fuses

The following safety precautions will prevent injury to personnel and damage to equipment. These are the MINIMUM safety precautions for replacing fuse.

1. Be sure the power is off in the circuit and the circuit is discharged before replacing a fuse.
2. Use an identical replacement fuse.
3. Remove any corrosion from the fuseholder before replacing the fuse.
4. Be certain the fuse properly fits the fuseholder.

How to fix the fuse

1. Turn the power off and discharge the circuit before removing a blown fuse.
2. Use a fusepuller (an insulated tool) when you remove a fuse from a clip-type fuseholder.
3. Remember, always replace the faulty fuse with one equal in size, amperage, rating and type.
4. Turn the main power switch back on and see if electricity is restored.

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